

ZOLL®



The Zoll AED Plus has a status indicator near the handle of the AED on the left hand side.



A green check means that the AED is ready to go, the batteries are ok and the pads are connected.



A red X means that the AED did not pass its self-test and it needs attention either by connecting the pads or replacing the battery

The Zoll AED Plus uses 10 consumer type 123A Photo Flash lithium manganese dioxide batteries. These can be purchased at most camera or electronic stores. The batteries should be replaced every 5 years or if the unit prompts you to replace them by beeping once every minute. After you replace all the batteries make sure to press the battery reset button in the battery compartment.



Battery replacement reset button

The Zoll AED Plus has 2 different adult pads that can be used with the unit. The one piece CPR-D-Padz has a 5 year shelf life and the two piece Stat-Padz have a 2 year shelf life. The two piece pediatric Pedi-Padz also has a 2 year shelf life.



Adult only CPR-D Padz

Adult Stat-Padz (could be used on a child if needed)

Pediatric Stat-Pad

